


Wooden Shower Stool

Installation And User Care Instructions



The Wooden Shower Stool is supplied with adjustable legs which allow 25mm incremental adjustment of the seat height between 410 and 535mm. Ensure all legs are set to the same height.

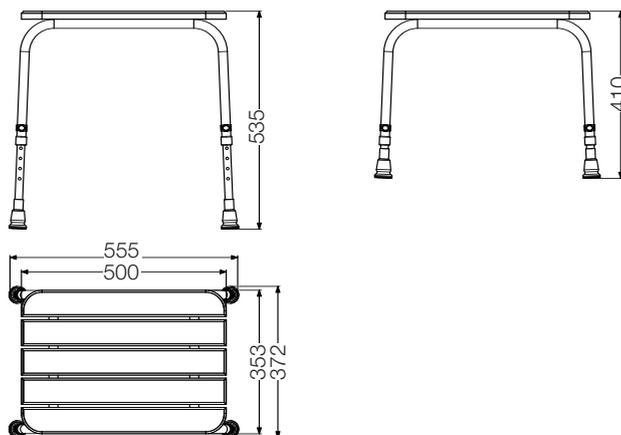

Keeping your Wooden Shower Stool Clean

This product should be cleaned periodically using normal domestic bathroom cleaning materials. After cleaning always rinse with water then wipe thoroughly with a damp cloth to remove any cleaning material residues before drying off.

Do not use strong or concentrated acidic or alkaline cleaning materials as these may discolour or otherwise damage the product.

AKW do not approve or recommend autoclave cleaning of any of the components due to the damage this can cause to the special engineering plastics used in the construction of this product.


Disinfecting

Where disinfecting is necessary, standard procedures for disinfecting equipment should be used taking due care and attention to the warnings concerning possible effects of aggressive chemicals on plastics.

Packaging

The Wooden Shower Stool is supplied in a polypropylene (PP) bag to keep the product unsoiled until installation, it may also be bulk packed in a cardboard box. Both are suitable for recycling operations and should be disposed of in accordance with the local recycling regulations.

Installer : Please ensure that this information is left with the end user of this product.

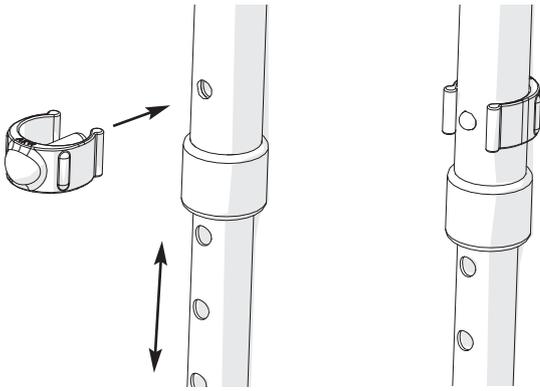

When adjusting the leg length align holes and ensure that the leg clip pin is clearly visible on the back of the leg. Ensure all legs are set to the same height.



NEVER exceed the weight capacity of 40 stone (254 kg).

BEWARE Leg height adjustment is intended for setting user preference to the seat height.



ALWAYS read the installation instructions and leave with the end user.



ALWAYS ensure that the legs are fitted securely with the adjustable leg clips firmly in place and the legs are of even length.



NEVER use if the stool is unstable in the shower.

NEVER use if the stool is damaged.

NEVER stand on the stool.

This may affect the strength of the shower stool and invalidate any warranty or stated weight capacity.



DO NOT place matches or cigarettes on or near this product.

The following materials are used in this product:

Nylon 6, TPE (Thermo Plastic Elastomer), Rubber Wood and Stainless Steel.

Failure to install this AKW Medicare product in accordance with supplied instructions, or the making of unauthorised modifications will invalidate any warranty and will affect product safety.

The measurements throughout these installation and user care instructions have been converted from metric to its equivalent in imperial, therefore all dimensions are approximate and subject to manufacturing tolerances. DLP Limited (AKW MediCare) does not make any warranty, expressed or implied, or accept any liability in connection with this information or its use. This information is furnished upon the condition that the person receiving it shall make his/her own tests to determine the suitability thereof for his/her particular purpose.

**For further details on this and other AKW products please contact your distributor
- see details on outer packaging**

AKW Medi-Care Limited, Pointon Way, Hampton Lovett, Droitwich Spa, WR9 0LR
Tel.+44 (0) 1905 823298 - Fax. +44 (0) 1905 823297